

Comfortmax

Sport Knee Brace

Part #67-100

BEFORE USING THIS DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THIS DEVICE.

INDICATIONS

The Comfortmax Sport knee brace provides stability and support for chronic knee pains and instability. Effective in treating mild injuries of the collateral ligaments, menisci, and patellar retinaculum.

APPLICATION

Remove the knee brace from its packaging and gently slip it up the leg and over the knee, making sure the sizing tag on the inside of the brace is at the top.

Position the knee brace so that the silicone padding fits comfortably over the patella/knee cap.



CLEANING INSTRUCTIONS

Remove hinges from brace and hand wash in cold water with mild detergent. Allow all parts to air dry before re-assembly. Do not dry clean, iron, or bleach any part of this brace, as it could cause damage and void the manufacturer's warranty.

NOTICE: While every effort has been made to obtain maximum strength, function, and durability, there is no guarantee that injury will be prevented through the use of this device.

Comfortland Medical, Inc.
709 AO Smith Rd.
Mebane, NC 27302
T 866.277.3135 | F 866.277.4412
www.comfortlandmed.com