

Comfortmax

Sport Ankle Brace

Part #67-200

BEFORE USING THIS DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THIS DEVICE.

INDICATIONS

The Comfortmax Sport ankle brace provides stability and support for chronic ankle pain and instability. Effective in treating mild injuries, arthrosis, ligament instability, for post cast removal.

APPLICATION

Remove the ankle brace from its packaging and gently slip it up the foot and over the foot, making sure the sizing tag on the inside of the brace is at the top.

Position the ankle brace so that the silicone padding fits comfortably on the sides of the ankle.



CLEANING INSTRUCTIONS

Hand wash in cold water with mild detergent. Allow to air dry. Do not dry clean, iron, or machine wash/dry.

NOTICE: While every effort has been made to obtain maximum strength, function, and durability, there is no guarantee that injury will be prevented through the use of this device.

Comfortland Medical, Inc.
709 AO Smith Rd.
Mebane, NC 27302
T 866.277.3135 | F 866.277.4412
www.comfortlandmed.com