Comfortland Suspension Sleeve

BEFORE USING THIS DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THIS DEVICE.

INDICATIONS

The Comfortland suspension sleeve is designed to be used in conjunction with Comfortland hinged knee braces to prevent brace migration and reduce skin irritation from long periods of wear. Effective in providing soft tissue compression to injured areas to speed the healing process.

CAUTION

The application instructions listed above are intended for the use of qualified professionals only. It is not recommended for patients to attempt to apply or adjust this device unless otherwise instructed to do so by an

orthopedic professional. Any questions regarding further adjustment or frequency/ duration of wear should be

directed to your orthopedic professional, as they know the individual condition for which this device was prescribed.

PREPARATION

The Comfortland suspension sleeve is a universally sized item, and requires no trimming/ adjustment before wear. The suspension sleeve can, however, be trimmed to a

specific length/ width for a more comfortable fit if needed (optional).

APPLICATION

To Prevent Brace Migration: Once the knee brace is properly fitted and adjusted, wrap the suspension sleeve over the brace, using the thumb loops on the sleeve to achieve the appropriate tension. Secure the sleeve to itself via the Velcro provided (Fig. A).

To Prevent Skin Irritation/ Add Soft Tissue Compression: Wrap the suspension sleeve around the desired area before the knee brace is applied, and secure the sleeve to itself via the Velcro provided. To be used in this manner, the suspension sleeve is most commonly applied to the thigh (Fig. B), the top of the calf (Fig. C), or directly over the knee itself (Fig. D). Once the suspension sleeve is properly secured, apply the knee brace over the sleeve as usual.









CLEANING INSTRUCTIONS

Hand wash in cold water with mild detergent. Air dry . Do not dry clean, iron, or bleach any part of this item, as it could cause damage and void the manufacturer's warranty.

NOTICE: While every effort has been made to obtain maximum strength, function, and durability, there is no guarantee that injury will be prevented through the use of this device.

Comfortland Medical, Inc. 709 AO Smith Rd. Mebane, NC 27302 T 866.277.3135 | F 866.277.4412 www.comfortlandmed.com