

CK-008 Comfortland Condylar Pad Sugg HCPC L2810 Instructions

Before using this device please read the following instructions completely and carefully, and also carefully follow any instructions given by your physician. Correct use is vital to the proper functioning of this device.

Indications:

•Knee and joint pain due to arthritis

- Post-op surgery
- Chronic knee pain

INSTRUCTION APPLICATIONS

First remove Condylar Pad(s) from packaging. Using hook and loop fastener, attach Condylar Pad(s) to inner hinge(s) of desired knee brace.

Fit and adjust knee brace as instructed, making sure that when knee brace is fitted Condylar Pad(s) fit snugly against the side(s) of the knee.



CLEANING INSTRUCTIONS

Hand wash with warm water and mild soap. Do not use bleach, disinfectants, or harsh detegents. Do not place in dishwasher or clothes washing machine. Allow to air dry, do not use clothes dryer.

> Comfortland Medical 709 AO Smith Road Mebane, NC 27302 T: 866-277-3135 F: 866-277-4412 www.comfortlandmed.com