

Comfortland Foot Drop Splint

BEFORE USING THIS DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THIS DEVICE.

INDICATIONS

The Foot drop splint provides stability and support for those suffering from foot drop. Effective in restoring a natural gait and proper biomechanics necessary for normal movement.

CAUTION

The application instructions listed above are intended for the use of qualified professionals only. It is not recommended for patients to attempt to apply or adjust this device unless otherwise instructed to do so by an orthopedic professional. Any questions regarding further adjustment or frequency/ duration of wear should be directed to your orthopedic professional, as they know the individual condition for which this device was prescribed.

PREPARATION

1. Begin by performing a “dry fit” to mark and trim any excess material from the footplate of the splint. Place your foot in the splint, making sure that the posterior strut of the splint makes full contact with the rear of the leg, and using a marker, trace around the toes. (Fig. A)
2. Using a pair of scissors, cut along the outline of the toes and use sandpaper to smooth the cut edge to prevent irritation.

APPLICATION

1. Loosen the top cuff strap and place your foot inside the splint, making sure that the posterior strut makes full contact with the back of the leg (Fig. B).
2. Wrap the cuff closure strap around the leg and secure to itself using the Velcro provided (Fig. C). The cuff strap should fit snug for maximum energy return.
3. Place your foot with the attached splint inside of your shoe as normal.



CLEANING INSTRUCTIONS

The foot drop splint can be wiped clean using a cloth with cold water and mild detergent. Air dry. Do not attempt to iron or machine dry any portion of this splint, as failure to do so can result in damage to this product and void the manufacturer's warranty.

NOTICE: While every effort has been made to obtain maximum strength, function, and durability, there is no guarantee that injury will be prevented through the use of this device.

Comfortland Medical, Inc.
709 AO Smith Rd.
Mebane, NC 27302
T 866.277.3135 | F 866.277.4412
www.comfortlandmed.com