

CK-802 Comfortland Universal Shoulder Abduction Pillow Instructions



Before using this device please read the following instructions completely and carefully, and also carefully follow any instruction sgiven by your physiscian. Correct use is vital to the proper functioning of this device.

Indications:

- Rotator cuff, elbow, and shoulder injuries
- Adhesive capsulitis/frozen shoulder
- Post-op and pre-op

Instructions:

1. First seperate and lay pieces on table or suitable flat surface to ease application (Item #1-Abduction pillow, Item #2-Waist strap, Item #3-Shoulder strap, Item #4-Arm/Humeral cuff, Item #5-Hand/wrist cuff). (Fig. A)
2. Place abduction pillow under arm. Rotate pillow to desired degree of abduction. (Fig. B)
3. Attach one end of waist strap to pillow, then wrap strap around patient waist and secure other end of strap to pillow as shown. (Fig. C)
4. With arm and shoulder in desired position, attach both the arm/humeral cuff and hand/wrist cuff. Buckle on hand/wrist strap should be on insidetoward patient's body. Close and secure both arm/humeral and hand/wrist straps. (Fig. D)
5. Attach velcro end of shoulder strap to back of pillow, then place strap over unaffected shoulder and attach to buckle on hand/wrist strap. Adjust to desired fit. (Fig. E)
6. If necessary readjust straps and closures for fit and support. (Fig F)

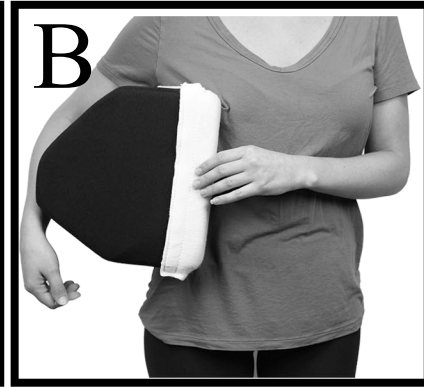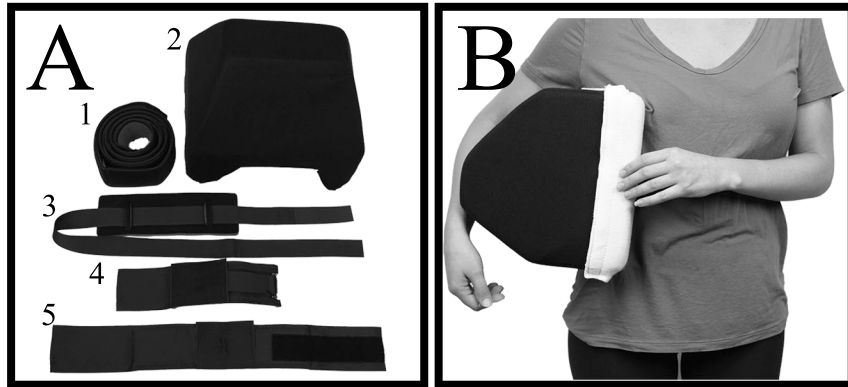

CLEANING INSTRUCTIONS

Surfaces and parts may be cleaned by damp cloth using mild soap and water. Air dry only.

Do not washine wash or dry, bleach, iron or dry clean.



Comfortland Medical
709 AO Smith Road
Mebane, NC 27302
T: 866-277-3135
F: 866-277-4412
www.comfortlandmed.com