

Comfortland Foam Collar

Fitting Instructions

Note: Your practitioner is familiar with your specific medical condition and is using this product as part of your total care. Make sure to follow their directions exactly, even if they do not match the directions below.

1) Begin by loosening the closure strap at the back of the collar. Place collar around the neck so that the closure strap is located at the back of the neck, and so that the indention in the top of the collar fits just under the chin.

2) Place one hand on the front of the neck to hold the collar in place and simultaneously attach the closure strap in the rear.

3) Tighten and re-adjust as needed. Make sure that the collar indention remains in line with the chin.

Cleaning and General Care:

Wash in cold water with mild detergent. Air dry. Do not dry clean, iron, or bleach.

www.comfortlandmed.com