



# Comfortmax Conductive Garments

## Operating Instructions

**Note:** Your practitioner is familiar with your specific medical condition and is using this product as part of your total care. Make sure to follow their directions exactly, even if they do not match the directions below.

1. Begin by cleaning the area to be treated thoroughly with soap and water. It is also recommended to remove all jewelry before treatment.
2. Take one set of wires from an approved electro stimulation device, and attach the active red pigtail to the treatment garment. Attach the black pigtail into a grounding electrode.
3. Slip the garment on the desired area of treatment. Place the grounding electrode on the body, away from the garment itself.
4. Apply the stimulation program as directed by your healthcare practitioner.
5. Once completed, unhook both the garment and the grounding electrode from the pigtails and let the garment air dry. Once dry, place the garment back in the bag provided.

### **Caution:**

*Do not apply to broken skin. The device and electrodes included are intended for single patient use only. If irritation occurs, discontinue use and consult your clinician. The electrodes provided should be replaced when they no longer adhere to the skin.*