

## CK-601 SILICONE GEL HEEL CUPS INSTRUCTIONS

## **Directions:**

- -Remove heel cups from packaging.
- -Loosen shoe laces.
- -Insert heel cups into heel of shoe.
- -insert feet into shoes, tighten and/or adjust lacing for fit and comfort.



-Only use with casual or dress shoes that have proper heel counters/heelcaps. Do not use with sandals, flip-flops, or slippers.

## Care Instructions:

- -Spot clean with mild detergent and damp cloth.
- -Do not soak or immerse fully in water.
- -Air dry, do not reinsert in shoe until dry.



Comfortland Medical 709 AO Smith Road Mebane, NC 27302 Phone: 866-277-3135

Fax: 866-277-4412

www.comfortlandmed.com