

CK-109
Comfortmax
Neoprene Hinged Knee
Sugg HCPC L1810/L1812
Instructions

Before using this device please read the following instructions completely and carefully, and also carefully follow any instructions given by your physician. Correct use is vital to the proper functioning of this device.

Indications:

- Sprains and strains of the knee
- Mild or moderate ligament pain
- Osgood Schlatter

INSTRUCTION APPLICATIONS

1. Remove knee brace from packaging.
2. Make sure that sizing tag is on the inside and at the front to ensure brace is in proper orientation.
3. Gently slip brace up over foot and lower leg to the knee.
4. Position knee brace so that hinges line up with sides of knee. Fit should be firm and secure, but not overly tight.



CLEANING INSTRUCTIONS

Hand wash with warm water and mild soap. Do not use bleach, disinfectants, or harsh detergents. Do not place in dishwasher or clothes washing machine. Allow to air dry, do not use clothes dryer.