



# Comfortland Comfort Abduction Control Joint Instructions

Part #CK-687  
SUGG HCPC L2624



Before using this device please read the following instructions completely and carefully, and also carefully follow any instruction given by your physician.  
Correct use is vital to the proper functioning of this device.

## To adjust for abduction and adduction:

- Loosen bolt on back side of ball joint with included Allen wrench
- Adjust ball joint to desired level of abduction or adduction, making sure to keep leg in midline
- Tighten bolt on back of ball joint with Allen wrench to lock in position
- Check and readjust if necessary

Comfortland Medical  
709 AO Smith Road  
Mebane, NC 27302  
T: 866-277-3135  
F: 866-277-4412  
[www.comfortlandmed.com](http://www.comfortlandmed.com)