

DS-15  
Comfortland Universal  
Clavicle Strap  
Sugg HCPC L3650  
Instructions

Before using this device please read the following instructions completely and carefully, and also carefully follow any instructions given by your physician. Correct use is vital to the proper functioning of this device.

## Indications:

- Clavicle fractures
- Acromioclavicular separation
- Posture control

## PREPARATION

1. Using the Velcro adjuster on the back of the splint, adjust the splint height to the desired degree (Fig. A). Loosen both shoulder straps to their widest point for easier application.

## APPLICATION

1. Place arms in each shoulder strap, making sure that the rear support is centered over the spine (Fig.B).

2. Simultaneously pull both shoulder straps until the clavicle splint fits snug and little movement is experienced (Fig. C).

3. While keeping tension on the shoulder straps, attach each strap to the Velcro provided. Tighten and re-adjust as needed. If applied properly, the Comfortland clavicle splint should resemble fig. D.



## CLEANING INSTRUCTIONS

Hand wash in cold water with mild detergent. Wipe the remainder of the brace with cold water. Air dry. Do not dry clean, iron, or bleach any part of this brace, as it could cause damage and void the manufacturer's warranty.



Comfortland Medical  
709 AO Smith Road  
Mebane, NC 27302  
T: 866-277-3135  
F: 866-277-4412  
[www.comfortlandmed.com](http://www.comfortlandmed.com)